

# Afinion HbA1c Quiz

Name: \_\_\_\_\_ Date: \_\_\_\_\_

1. What is the Afinion HbA1c 's measuring range?

- a. 2 – 14%
- b. 6 – 20 mg/dl
- c. 4 – 15%
- d. 2 – 18 mg/dl

2. What are acceptable sample types for the Afinion HbA1c assay?

- a. Capillary blood (finger stick)
- b. Venous whole blood (EDTA, heparin or citrate)
- c. Serum
- d. a and b

3. How long are Afinion HbA1c controls stable for once the vials are opened?

- a. 30 days
- b. 60 days
- c. 90 days
- d. Until the expiration date printed on the box and vials

4. What is the sample size for the Afinion HbA1c assay?

- a. 1.0 uL
- b. 1.5 uL
- c. 2.0 uL
- d. 5.0 uL

5. True or False: Afinion HbA1c controls can be stored at room temperature (59-77 F)

\_\_\_\_\_

6. What is the Afinion's hemoglobin measuring range?

- a. 2-14 %
  - b. 6-20 mg/dl
  - c. 4-15%
  - d. 2-18 mg/dl
- 

7. Venous whole blood samples can be stored refrigerated for how many days?

- a. 1 day
- b. 5 days
- c. 10 days
- d. Samples cannot be stored refrigerated

8. How often should Afinion HbA1c controls be run as per the manufacturer's recommendations?

- a. At least every 30 days
- b. With each new shipment of HbA1c test kits
- c. With each new lot of HbA1c test kits
- d. All of the above

9. True or False: Afinion HbA1c test cartridges can be used immediately after removing them from the refrigerator?

\_\_\_\_\_

10. How long are Afinion HbA1c test cartridges stable for if stored at room temperature (59-77 F)?

- a. 30 days
- b. 60 days
- c. 90 days
- d. Until the expiration date printed on the box and foil pouch